BVBA REVILAX SPRL - Kokkelstraat 15 - B-3680
Neeroeteren
Tel.: +32 89 473 460 - Fax: +32 89 704 908 - www.revilax.be

RPR TONGEREN - BTW BE 0453.236.953 - ING 335-0112980-21 - IBAN BE 17 3350 1129 8021 - BIC (SWIFT) BBRUBEBB

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Make sure there is enough free space behind and in front of the chair.

Do not position the chair nearby heat sources. Do not cover the adaptor.

Only use the adaptor that has been delivered with your Revilax® chair.

Make sure the adaptor stands free and not under the chair.

You may only stand up if the leg support has been completely closed. The leg support is meant to support your legs only and cannot be charged additionally.

The lift function is conceived to facilitate the movement of standing up and getting seated. Important: the lift position is NO seating position! Use your chair indoor and in a dry environment. Cleaning with a humid cloth.

Should you accidently spill fluid over the chair, immediately disconnect the adaptor and wait until the electric parts are completely dry. If necessary wait up to 24 hours before connecting the chair again.

Your Revilax® chair is no toy. Operation by children is only authorised when supervised. If supervision is not possible: disconnect the chair.

If wires are damaged, disconnect the adaptor and call the Revilax® after sales service.

The maximum supply power is 240V.